

## **Test Definition: METQU**

Methaqualone, Urine

Reporting Title: Methaqualone, U

Performing Location: Rochester

### Specimen Requirements:

Container/Tube: Plastic, 60-mL urine bottle

Specimen Volume: 20 mL Collection Instructions:

- 1. Collect a random urine specimen.
- 2. No preservative. Additional Information:
- 1. For situations where chain-of-custody is required, a Chain-of-Custody Kit (Supply T282) is available. For chain-of-custody information, see COCH/9426 Chain-of-Custody Processing.
- 2. Additional drug panels and specific requests are available. Call Mayo Medical Laboratories at 800-533-1710 or 507-266-5700.

Specimen Type	Temperature	Time
Urine	Refrigerated (preferred)	14 days
	Frozen	14 days
	Ambient	72 hours

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
24038	Immunoassay Screen	Numeric	ng/mL	3787-9
24039	Methaqualone- by GC/MS	Alphanumeric	ng/mL	16244-6
24040	Interpretation	Alphanumeric		69050-3
24043	Chain of Custody	Numeric		In Process

**CPT Code:** 1 × 80299

#### **Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
ADULT	Adulterants Survey, U			Yes	No



# **Test Definition: METQU**

Methaqualone, Urine

### **Result Codes for Reflex Tests:**

Test ID	Result ID	Reporting Name	Туре	Unit	LOINC®
ADULT	20606	Creatinine, U	Numeric	mg/dL	In Process
ADULT	22312	Specific Gravity	Numeric		5811-5
ADULT	23509	рН	Numeric		2756-5
ADULT	23511	Oxidants	Alphanumeric		In Process
ADULT	23510	Nitrites	Alphanumeric	mcg/mL	32710-6
ADULT	30914	Comment	Alphanumeric		48767-8

#### **Reference Values:**

Negative

Positives are reported with a quantitative GC-MS result. Cutoff concentrations: IMMUNOASSAY SCREEN <300 ng/mL

METHAQUALONE BY GC-MS <100 ng/mL