

Test Definition: FPSAU

Prostate-Specific Antigen, Ultrasensitive

Reporting Title: PSA, Ultrasensitive **Performing Location:** Labcorp Burlington

Specimen Requirements:

Draw blood in a red-top tube or serum gel tube. Spin down and send 0.5 mL of serum in plastic screw-capped tube, shipped frozen.

Specimen Type	Temperature	Time
Serum	Frozen (preferred)	14 days
	Ambient	7 days
	Refrigerated	7 days

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
Z0983	PSA, Ultrasensitive	Numeric	ng/mL	

CPT Code: 1 × 84153

Reference Values:

0.0 0-4.00 ng/mL

Roche ECLIA methodology

According to the American Urological Association, PSA should be undetectable after radical prostatectomy. A PSA of less than 0.50 ng/mL (or undetectable) is not likely to be associated with disease recurrence within five years of treatment. Values obtained with different assay methods or kits cannot be used interchangeably. Results cannot be interpreted as absolute evidence of the presence or absence of malignant disease.

Test Performed by: LabCorp Burlington 1447 York Court Burlington, NC 27215