

Test Definition: UREDF

Reducing Substance, Feces

Reporting Title: Reducing Substance, F

Performing Location: Rochester

Specimen Requirements:

Container/Tube: Stool container (Supply T288)

Specimen Volume: 3 g Collection Instructions:

- 1. Collect a loose, random stool collection.
- 2. See Stool Collection Information Sheet in Special Instructions.
- 3. If collecting from an infant, use nonabsorbent diapers to collect the specimen. If stool is collected in a disposable diaper, the solid and liquid stool must be combined. The liquid portion of the stool can be aspirated from the diaper using a syringe or alternatively can be expressed from the diaper into a cup. If a diaper has gel absorption, a cotton ball will be necessary to collect the liquid portion of the stool.
- 4. It is critical that urine is not also mixed with the specimen. Urine will interfere with the results.

Specimen Type	Temperature	Time
Fecal	Frozen	7 days

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
6215	Reducing Substance, F	Alphanumeric		11060-1

CPT Code: 1 × 84376

Reference Values:

Negative or trace