

TEST UPDATE REFERRAL

NOTIFICATION DATE: May 29, 2012 **EFFECTIVE DATE:** Immediately

FLUNITRAZEPAM (ROHYPNOL), SERUM

Test ID: FFLRO Secondary ID: 91795

EXPLANATION OF CHANGE: Test ID FFLRO, referred to Medtox Laboratories, Inc., will reflect the following updates effective immediately.

CURRENT PUBLISHED NAME: Flunitrazepam (Rohypnol), Serum

NEW PUBLISHED NAME: Flunitrazepam, Confirmation, Serum

CURRENT CPT CODE: 82542

NEW CPT CODE: 80154

CURRENT SPECIMEN REQUIREMENTS:

Submit only 1 of the following specimens:

Serum

Draw blood in a plain, red-top tube(s). (Serum gel tube is not acceptable.) Spin down and send 10 mL of serum refrigerated.

Note: 1. Indicate serum on request from.

2. Label specimen appropriately (serum).

Plasma

Draw blood in a green-top, sodium heparin tube(s). (Plasma gel tube is not acceptable.) Spin down and send 10 mL of sodium heparin plasma refrigerated.

Note: 1. Indicate plasma on request form.

2. Label specimen appropriately (plasma).

NEW SPECIMEN REQUIREMENTS:

Submit only 1 of the following specimens:

Serum

Draw blood in a plain, red-top tube(s). (Serum gel tube is not acceptable.) Spin down and send 5 mL of serum refrigerated.

Note: 1. Label specimen appropriately (serum).

Plasma

Draw blood in a green-top, sodium heparin tube(s). (Plasma gel tube is not acceptable.) Spin down and send 5 mL of sodium heparin plasma refrigerated.

Note: 1. Label specimen appropriately (plasma).

QUESTIONS: Contact Mary Erath, MML Laboratory Technologist Resource Coordinator Telephone: 800-533-1710