

# **Test Definition: COKEU**

Cocaine Confirmation, Urine

Reporting Title: Cocaine Confirmation, U

Performing Location: Rochester

### **Specimen Requirements:**

Collection Container/Tube: Plastic urine container Submission Container/Tube: Plastic, 60-mL urine bottle

Specimen Volume: 20 mL Collection Instructions:

- 1. Collect a random urine specimen.
- 2. Submit 20 mL in 1 plastic bottle.
- 3. No preservative. Additional Information:
- 1. No specimen substitutions.
- 2. No STATS are accepted for this procedure.
- 3. For situations where chain-of-custody is required, a Chain-of-Custody Kit (Supply T282) is available. For chain-of-custody information, see COCH/9426 Chain-of-Custody Processing.
- 4. Additional drug panels and specific requests are available. Call Mayo Medical Laboratories at 800-533-1710 or 507-266-5700.

Specimen Type	Temperature	Time	
Urine	Refrigerated (preferred)	14 days	
	Ambient	72 hours	
	Frozen	14 days	

### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
2903	Immunoassay Screen	Numeric		
2922	Cocaine-by GC/MS	Alphanumeric	ng/mL	
2902	Benzoylecgonine-by GC/MS	Alphanumeric	ng/mL	
20671	Interpretation	Alphanumeric		
28080	Chain of Custody	Numeric		

**CPT Code:** 1 × 82520



# **Test Definition: COKEU**

Cocaine Confirmation, Urine

#### **Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
ADULT	Adulterants Survey, U			Yes	No

## **Result Codes for Reflex Tests:**

Test ID	Result ID	Reporting Name	Туре	Unit	LOINC®
ADULT	20606	Creatinine, U	Numeric	mg/dL	
ADULT	22312	Specific Gravity	Numeric		
ADULT	23509	рН	Numeric		
ADULT	23511	Oxidants	Alphanumeric		
ADULT	23510	Nitrites	Alphanumeric	mcg/mL	
ADULT	30914	Comment	Alphanumeric		

### **Reference Values:**

Negative
Positives are reported with a quantitative GC-MS result.
Cutoff concentrations:
IMMUNOASSAY SCREEN
<150 ng/mL

COCAINE BY GC-MS <50 ng/mL

BENZOYLECGONINE BY GC-MS <50 ng/mL