

Pain Clinic Drug Screen, Urine

Reporting Title: Pain Clinic Drug Screen, U

Performing Location: Rochester

#### **Specimen Requirements:**

Collection Container/Tube: Plastic urine container Submission Container/Tube: Plastic, 60-mL urine bottle

Specimen Volume: 30 mL Collection Instructions:

- 1. Collect a random urine specimen.
- 2. Submit 30 mL in 1 plastic bottle.
- 3. No preservative. Additional Information:
- No specimen substitutions.
- 2. No STATS are accepted for this procedure.
- 3. For situations where chain-of-custody is required, a Chain-of-Custody Kit (Supply T282) is available. For chain-of-custody information, see COCH/9426 Chain-of-Custody Processing.
- 4. Additional drug panels and specific requests are available. Call Mayo Medical Laboratories at 800-533-1710 or 507-266-5700.

Specimen Type	Temperature	Time
Urine	Refrigerated (preferred)	14 days
	Ambient	72 hours
	Frozen	14 days

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
6541	Immunoassay Screen	Numeric		
	Also used by tests: OPATU			
6543	Codeine-by GC/MS	Alphanumeric	ng/mL	
	Also used by tests: OPATU			
21177	Hydrocodone-by GC/MS	Alphanumeric	ng/mL	
	Also used by tests: OPATU			
21179	Hydromorphone-by GC/MS	Alphanumeric	ng/mL	
	Also used by tests: OPATU			



Pain Clinic Drug Screen, Urine

Result ID	Reporting Name	Туре	Unit	LOINC®
21178	Oxycodone-by GC/MS	Alphanumeric	ng/mL	
	Also used by tests: OPATU			
6542	Morphine-by GC/MS	Alphanumeric	ng/mL	
	Also used by tests: OPATU			
21180	Interpretation	Alphanumeric		
	Also used by tests: OPATU			
21181	Chain of Custody	Numeric		
	Also used by tests: OPATU			
31260	Drugs detected:	Alphanumeric		
	Also used by tests: PDSU			
45529	Suspect Drug	Alphanumeric		
	Also used by tests: PDSU			
31262	Chain of Custody	Numeric		
	Also used by tests: PDSU			
30909	Alcohol	Numeric	mg/dL	
	Also used by tests: IDOAU, CDAU, CDAU7, CDAU1			
2573	Amphetamines	Numeric	ng/mL	
	Also used by tests: CDAU, CDAU5, CDAU7, CDAU3, CDAU1, CDAU2			
2574	Barbiturates	Numeric	ng/mL	
	Also used by tests: CDAU1, IDOAU, CDAU7, CDAU3, CDAU, CDAU2			
2575	Benzodiazepines	Numeric	ng/mL	
	Also used by tests: CDAU3, CDAU7, CDAU			
21652	Cocaine	Numeric	ng/mL	
	Also used by tests: CDAU2, CDAU3, CDAU7, CDAU5, CDAU1, CDAU			
4435	Methadone	Numeric	ng/mL	
	Also used by tests: CDAU2, CDAU7, CDAU1			
2577	Opiates	Numeric	ng/mL	
	Also used by tests: CDAU1, CDAU, CDAU3, CDAU2, CDAU7, IDOAU, CDAU5			

Pain Clinic Drug Screen, Urine

Result ID	Reporting Name	Туре	Unit	LOINC®
2578	Phencyclidine  Also used by tests: CDAU1, CDAU5, CDAU3, CDAU2, CDAU, IDOAU, CDAU7	Numeric	ng/mL	
4436	Propoxyphene  Also used by tests: CDAU2, CDAU7, CDAU1	Numeric	ng/mL	
2664	Tetrahydrocannabinol  Also used by tests: CDAU5, CDAU3, CDAU, CDAU7	Numeric	ng/mL	
20672	Chain of Custody  Also used by tests: CDAU1, IDOAU, CDAU5, CDAU2, CDAU, CDAU3, CDAU7	Numeric		

### **Components:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
OPATU	Opiates, U	1	83925	Yes	Yes
PDSU	Drug Screen, Prescription/OTC, U	1	80100	Yes	Yes
Billing only	Conf Drug Abuse Sur 12,U	1	80104		
CDAU7	Confirmed Drug Abuse Survey 12, U	1	80104	Yes	Yes

### **Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
ADULT	Adulterants Survey, U			Yes	No
AMPHU	Amphetamines, U	2	82145	No	Yes
COKEU	Cocaine Confirmation, U	1	82520	No	Yes
BENZU	Benzodiazepine Confirmation, U	1	80154	No	Yes
BARBU	Barbiturate Confirmation, U	1	82205	No	Yes
MTDNU	Methadone Confirmation, U	1	83840	No	Yes
PCPU	Phencyclidine Confirmation, U	1	83992	No	Yes

Pain Clinic Drug Screen, Urine

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
MSPR	Drug of Abuse, Propoxyphene Conf, U	2	80299	No	Yes
THCU	Tetrahydrocannabinol Conf, U	1	80299	No	Yes
ЕТОН	Ethanol, U	1	82055	No	No

### **Result Codes for Reflex Tests:**

Test ID	Result ID	Reporting Name	Туре	Unit	LOINC®
ADULT	20606	Creatinine, U	Numeric	mg/dL	
ADULT	22312	Specific Gravity	Numeric		
ADULT	23509	pH	Numeric		
ADULT	23511	Oxidants	Alphanumeric		
ADULT	23510	Nitrites	Alphanumeric	mcg/mL	
ADULT	30914	Comment	Alphanumeric		
AMPHU	6538	Immunoassay Screen	Numeric		
AMPHU	2934	Amphetamine-by GC/MS	Alphanumeric	ng/mL	
AMPHU	29278	Phentermine-by GC/MS	Alphanumeric	ng/mL	
AMPHU	2550	Methamphetamine-by GC/MS	Alphanumeric	ng/mL	
AMPHU	29279	Pseudoephedrine/Ephedrine-by GC/MS	Alphanumeric	ng/mL	
AMPHU	29280	MDA (Ecstasy metabolite)-by GC/MS	Alphanumeric	ng/mL	
AMPHU	29281	MDMA (Ecstasy)-by GC/MS	Alphanumeric	ng/mL	
AMPHU	21197	Interpretation	Alphanumeric		
AMPHU	20641	Chain of Custody	Numeric		
BARBU	6668	Immunoassay Screen	Numeric		
BARBU	21442	Butalbital-by GC/MS	Alphanumeric	ng/mL	
BARBU	20642	Amobarbital-by GC/MS	Alphanumeric	ng/mL	
BARBU	20643	Pentobarbital-by GC/MS	Alphanumeric	ng/mL	
BARBU	20644	Secobarbital-by GC/MS	Alphanumeric	ng/mL	
BARBU	20645	Phenobarbital-by GC/MS	Alphanumeric	ng/mL	
BARBU	20651	Interpretation	Alphanumeric		
BARBU	20647	Chain of Custody	Numeric		



Pain Clinic Drug Screen, Urine

Test ID	Result ID	Reporting Name	Туре	Unit	LOINC®
BENZU	6670	Immunoassay Screen	Numeric		
BENZU	21441	Nordiazepam-by GC/MS	Alphanumeric	ng/mL	
BENZU	20655	Oxazepam-by GC/MS	Alphanumeric	ng/mL	
BENZU	20656	Lorazepam-by GC/MS	Alphanumeric	ng/mL	
BENZU	20657	Temazepam-by GC/MS	Alphanumeric	ng/mL	
BENZU	20658	OH-Ethyl-Flurazepam-by GC/MS	Alphanumeric	ng/mL	
BENZU	20659	7-NH-Clonazepam-by GC/MS	Alphanumeric	ng/mL	
BENZU	20670	Alpha OH-Alprazolam-by GC/MS	Alphanumeric	ng/mL	
BENZU	20660	7-NH-Flunitrazepam-by GC/MS	Alphanumeric	ng/mL	
BENZU	20661	Alpha OH-Triazolam-by GC/MS	Alphanumeric	ng/mL	
BENZU	20663	Interpretation	Alphanumeric		
BENZU	20664	Chain of Custody	Numeric		
COKEU	2903	Immunoassay Screen	Numeric		
COKEU	2922	Cocaine-by GC/MS	Alphanumeric	ng/mL	
COKEU	2902	Benzoylecgonine-by GC/MS	Alphanumeric	ng/mL	
COKEU	20671	Interpretation	Alphanumeric		
COKEU	28080	Chain of Custody	Numeric		
ETOH	30927	Immunoassay Screen	Numeric	mg/dL	
ETOH	30925	Ethanol-by GC-FID	Numeric		
ETOH	24032	Interpretation	Alphanumeric		
ETOH	24035	Chain of Custody	Numeric		
MSPR	11148	GC/MS Confirmation - Propoxyphene	Alphanumeric		
MSPR	33142	Propoxyphene	Alphanumeric	ng/mL	
MSPR	33143	Norpropoxyphene	Alphanumeric	ng/mL	
MSPR	33144	Comment	Alphanumeric		
MTDNU	21105	Immunoassay Screen	Numeric		
MTDNU	56028	EDDP-by GC-MS	Alphanumeric	ng/mL	
MTDNU	83129	Methadone-by GC-MS	Alphanumeric	ng/mL	
MTDNU	21107	Interpretation	Alphanumeric		
MTDNU	21110	Chain of Custody	Numeric		
PCPU	6672	Immunoassay Screen	Numeric		



Pain Clinic Drug Screen, Urine

Test ID	Result ID	Reporting Name	Туре	Unit	LOINC®
PCPU	6673	Phencyclidine-by GC/MS	Alphanumeric	ng/mL	
PCPU	21192	Interpretation	Alphanumeric		
PCPU	21193	Chain of Custody	Numeric		
THCU	2449	Immunoassay Screen	Numeric		
THCU	2497	THC Carboxylic Acid-by GC/MS	Alphanumeric	ng/mL	
THCU	21186	Interpretation	Alphanumeric		
THCU	21187	Chain of Custody	Numeric		

#### **Reference Values:**

Negative

Screening cutoff concentrations

Alcohol: 30 mg/dL

Amphetamines: 500 ng/mL Barbiturates: 200 ng/mL Benzodiazepines: 200 ng/mL

Cocaine (benzoylecgonine-cocaine metabolite): 150 ng/mL

Ethanol: 30 mg/dL Methadone: 300 ng/mL Opiates: 300 ng/mL Phencyclidine: 25 ng/mL Propoxyphene: 300 ng/mL

Tetrahydrocannabinol carboxylic acid: 20 ng/mL

This report is intended for use in clinical monitoring or management of patients. It is not intended for use in

employment-related testing.