

Reporting Title: Drugs of Abuse, Clin, DAU10, U
Performing Location: New England

Specimen Requirements:

Container/Tube: Plastic, 60-mL urine container (Supply T313)

Specimen Volume: 30 mL

Collection Instructions:

1. Collect a random urine specimen.
2. No preservative.

| Specimen Type | Temperature | Time |
|---------------|--------------------------|---------|
| Urine | Refrigerated (preferred) | 14 days |
| | Ambient | 14 days |
| | Frozen | 14 days |

Result Codes:

| Result ID | Reporting Name | Type | Unit | LOINC® |
|-----------|--|---------|-------|--------|
| 11164 | Amphetamines Also used by tests: DAAMP, DAU9, DAU7, DAU8 | Numeric | ng/mL | 8150-5 |
| 11301 | Barbiturates Also used by tests: DAU7, DAU9, DABAR, DAU8 | Numeric | ng/mL | 9426-8 |
| 11572 | Benzodiazepines Also used by tests: DAU8, DABZO, DAU9, DAU7 | Numeric | ng/mL | 9428-4 |
| 11161 | Cocaine Metabolite Also used by tests: DAU8, DAUCO, DAU9, DAU7 | Numeric | ng/mL | 3394-4 |
| 11165 | Methadone Also used by tests: DMETH, DAU9 | Numeric | ng/mL | 3774-7 |
| 11570 | Opiates Also used by tests: DAU8, DAU9, DAU7 | Numeric | ng/mL | 8220-6 |
| 11573 | Phencyclidine Also used by tests: DAU8, DPCP, DAU9, DAU7 | Numeric | ng/mL | 3937-0 |

| Result ID | Reporting Name | Type | Unit | LOINC® |
|-----------|--|--------------|-------|--------|
| 11166 | Propoxyphene Also used by tests: DPRP, DAU9 | Numeric | ng/mL | 3545-1 |
| 11162 | Tetrahydrocannabinols Also used by tests: DAU7, DAU8, DTHC, DAU9 | Numeric | ng/mL | 3530-3 |
| 501206 | Ethanol Also used by tests: DAU8 | Numeric | mg/dL | 5645-7 |
| 200209 | Comment Also used by tests: DAU9, DAU7, DAU8 | Alphanumeric | | N/A |

CPT Code Information:

80104-Drug screen, qualitative, multiple drug class method
 82055-Ethanol
 G0431-Drug screen, qualitative, multiple drug class method-government payers (if appropriate)
 80154-Benzodiazepines (if appropriate)
 80299-Propoxyphene (if appropriate)
 80299-Tetrahydrocannabinol carboxylic acid (if appropriate)
 82145-Amphetamines (if appropriate)
 82205-Barbiturates (if appropriate)
 82520-Cocaine (if appropriate)
 83840-Methadone (if appropriate)
 83925-Opiates (if appropriate)
 83992-Phencyclidine (if appropriate)

Reflex Tests:

| Test ID | Reporting Name | CPT Units | CPT Code | Always Performed | Orderable Separately |
|---------|-------------------------------------|-----------|----------|------------------|----------------------|
| MSOP | Drug of Abuse, Opiate Conf, U | 1 | 83925 | No | No |
| DAUAM | Drug of Abuse, Amphetamine Conf, U | 2 | 82145 | No | Yes |
| DAUBZ | Drug of Abuse, Benzo Conf, U | 1 | 80154 | No | Yes |
| MSBR | Drug of Abuse, Barbiturate Conf, U | 1 | 82205 | No | Yes |
| MSCO | Drug of Abuse, Cocaine Conf, U | 1 | 82520 | No | Yes |
| MSMT | Drug of Abuse, Methadone Conf, U | 1 | 83840 | No | Yes |
| MSPCP | Drug of Abuse, Phencyclidine Conf U | 1 | 83992 | No | Yes |

| Test ID | Reporting Name | CPT Units | CPT Code | Always Performed | Orderable Separately |
|---------|-------------------------------------|-----------|----------|------------------|----------------------|
| MSPR | Drug of Abuse, Propoxyphene Conf, U | 2 | 80299 | No | Yes |
| MSTHC | Drug of Abuse, THC Conf, U | 1 | 80299 | No | Yes |

Result Codes for Reflex Tests:

| Test ID | Result ID | Reporting Name | Type | Unit | LOINC® |
|---------|-----------|-------------------------------------|--------------|-------|------------|
| MSOP | 11141 | Confirmation - Opiates | Alphanumeric | | 17384-9 |
| MSOP | 200210 | Codeine | Alphanumeric | ng/mL | 16250-3 |
| MSOP | 200211 | Hydrocodone | Alphanumeric | ng/mL | 16252-9 |
| MSOP | 200212 | Hydromorphone | Alphanumeric | ng/mL | 16998-7 |
| MSOP | 200214 | Morphine | Alphanumeric | ng/mL | 16251-1 |
| MSOP | 200213 | Oxycodone | Alphanumeric | ng/mL | 16249-5 |
| MSOP | 200289 | Oxymorphone | Alphanumeric | ng/mL | 17395-5 |
| DAUAM | 501207 | GC/MS Confirmation - Amphetamine | Alphanumeric | | 16234-7 |
| DAUAM | 501208 | Amphetamine | Alphanumeric | ng/mL | 16234-7 |
| DAUAM | 501209 | Methamphetamine | Alphanumeric | ng/mL | 16235-4 |
| DAUAM | 500197 | MDMA (Ecstasy) | Alphanumeric | ng/mL | 18358-2 |
| DAUAM | 501210 | MDA (Ecstasy Metabolite) | Alphanumeric | ng/mL | 27409-2 |
| DAUAM | 501211 | Phentermine | Alphanumeric | ng/mL | 20557-5 |
| DAUAM | 501212 | Pseudoephedrine/Ephedrine | Alphanumeric | ng/mL | 26785-6 |
| DAUAM | 501213 | Phenylpropanolamine | Alphanumeric | ng/mL | 17443-3 |
| DAUAM | 501214 | Comment | Alphanumeric | | In Process |
| DAUBZ | 501215 | GC/MS Confirmation - Benzodiazepine | Alphanumeric | | 20412-3 |
| DAUBZ | 501216 | Lorazepam | Alphanumeric | ng/mL | 17088-6 |
| DAUBZ | 501217 | Nordiazepam | Alphanumeric | ng/mL | 16228-9 |
| DAUBZ | 501218 | Oxazepam | Alphanumeric | ng/mL | 16201-6 |
| DAUBZ | 501219 | Temazepam | Alphanumeric | ng/mL | 20559-1 |
| DAUBZ | 501220 | OH-Ethyl-Flurazepam | Alphanumeric | ng/mL | 20532-8 |
| DAUBZ | 501221 | 7-NH-Clonazepam | Alphanumeric | ng/mL | 51776-3 |
| DAUBZ | 501222 | 7-NH-Flunitrazepam | Alphanumeric | ng/mL | 51777-1 |



| Test ID | Result ID | Reporting Name | Type | Unit | LOINC® |
|---------|-----------|------------------------------------|--------------|-------|------------|
| DAUBZ | 501223 | Alpha OH-Alprazolam | Alphanumeric | ng/mL | 16348-5 |
| DAUBZ | 501224 | Alpha-OH-Triazolam | Alphanumeric | ng/mL | 49876-6 |
| MSBR | 501225 | Comment | Alphanumeric | | In Process |
| MSBR | 2837 | GC/MS Confirmation - Barbiturate | Alphanumeric | | 20411-5 |
| MSBR | 501226 | Amobarbital | Alphanumeric | ng/mL | 16239-6 |
| MSBR | 501227 | Butalbital | Alphanumeric | ng/mL | 16237-0 |
| MSBR | 501228 | Pentobarbital | Alphanumeric | ng/mL | 16240-4 |
| MSBR | 501229 | Phenobarbital | Alphanumeric | ng/mL | 16241-2 |
| MSBR | 501230 | Secobarbital | Alphanumeric | ng/mL | 16238-8 |
| MSCO | COCAC | GC/MS Confirmation - Cocaine | Alphanumeric | | 20519-5 |
| MSCO | COCUC | Cocaine | Alphanumeric | ng/mL | In Process |
| MSCO | BEUC | Benzoyllecgonine | Alphanumeric | ng/mL | 16226-3 |
| MSCO | CMT94 | Comment | Alphanumeric | | N/A |
| MSMT | 11147 | GC/MS Confirmation - Methadone | Alphanumeric | | 16246-1 |
| MSMT | 501240 | Methadone | Alphanumeric | ng/mL | 16246-1 |
| MSMT | 501241 | Comment | Alphanumeric | | In Process |
| MSPCP | 11146 | GC/MS Confirmation - Phencyclidine | Alphanumeric | | 16254-5 |
| MSPCP | 501238 | Phencyclidine | Alphanumeric | ng/mL | 16254-5 |
| MSPCP | 501239 | Comment | Alphanumeric | | In Process |
| MSPR | 11148 | GC/MS Confirmation - Propoxyphene | Alphanumeric | | 16242-0 |
| MSPR | 33142 | Propoxyphene | Alphanumeric | ng/mL | 16242-0 |
| MSPR | 33143 | Norpropoxyphene | Alphanumeric | ng/mL | 17377-3 |
| MSPR | 33144 | Comment | Alphanumeric | | 8251-1 |
| MSTHC | 11145 | GC/MS Confirmation - THC | Alphanumeric | | 20413-1 |
| MSTHC | 501233 | THC Carboxylic Acid | Alphanumeric | ng/mL | 3530-3 |
| MSTHC | 501234 | Comment | Alphanumeric | | In Process |



Reference Values:

Negative

The specific drug identified will be reported.

EMIT cutoff concentrations

Amphetamines: 500 ng/mL

Barbiturates: 200 ng/mL

Benzodiazepines: 200 ng/mL

Cocaine (benzoylecgonine-cocaine metabolite): 150 ng/mL

Methadone: 300 ng/mL

Opiates: 300 ng/mL

Phencyclidine: 25 ng/mL

Propoxyphene: 300 ng/mL

THC: 50 ng/mL

Ethanol: 10 mg/dL

Results of this test are confirmed and should be considered definitive. Specimens are normally kept for 2 weeks after Mayo Medical Laboratories New England receipt date.