

Test Definition: NORT

Nortriptyline, Serum

Reporting Title: Nortriptyline, S Performing Location: Rochester

Specimen Requirements:

Container/Tube: Red top Specimen Volume: 3 mL Collection Instructions:

- 1. Draw specimen immediately before next scheduled dose (minimum 12 hours after last dose).
- 2. Serum must be separated from cells within 2 hours of draw.

Specimen Type	Temperature	Time
Serum Red	Refrigerated (preferred)	14 days
	Ambient	14 days
	Frozen	14 days

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
81858	Nortriptyline, S	Numeric	ng/mL	3872-9

CPT Code: 1 × 80182

Reference Values:

Therapeutic concentration: 70-170 ng/mL Toxic concentration: > or =300 ng/mL

Note: Therapeutic ranges are for specimens drawn at trough (ie, immediately before next scheduled dose). Levels may

be elevated in non-trough specimens.