

Reporting Title: Doxepin and Nordoxepin, S
Performing Location: Rochester

Specimen Requirements:

Container/Tube: Red top

Specimen Volume: 3 mL

Collection Instructions:

1. Draw specimen immediately before next scheduled dose (or at a minimum 12 hours after last dose).
2. Serum must be separated from cells within 2 hours of draw

Specimen Type	Temperature	Time
Serum Red	Refrigerated (preferred)	14 days
	Ambient	14 days
	Frozen	14 days

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
9301	Doxepin	Numeric	ng/mL	3579-0
2464	Nordoxepin	Numeric	ng/mL	3862-0
2470	Doxepin and Nordoxepin	Numeric	ng/mL	3582-4

Components:

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
Billing only	Doxepin	1	80166		
Billing only	Quantitation of drug	1	80299		

Reference Values:

Therapeutic concentration (doxepin + nordoxepin): 50-150 ng/mL

Toxic concentration (doxepin + nordoxepin): > or =300 ng/mL