

# **Test Definition: LORAZ**

Lorazepam (Ativan), Serum

Reporting Title: Lorazepam (Ativan)
Performing Location: Medtox Laboratories

## **Specimen Requirements:**

Submit only 1 of the following specimens:

#### Plasma

Draw blood in a green-top sodium heparin tube(s). (Plasma gel tube is not acceptable.) Spin down and send 2 mL of plasma refrigerated.

Note: 1. Indicate plasma on request form.

- 2. Label specimen appropriately (plasma).
  - 3. Trough levels are most reproducible.

#### Serum

Draw blood in a plain, red-top tube(s). (Serum gel tube is not acceptable.) Spin down and send 2 mL of serum refrigerated.

Note: 1. Indicate serum on request form.

- 2. Label specimen appropriately (serum).
- 3. Trough levels are most reproducible.

Specimen Type	Temperature	Time	
Varies	Refrigerated (preferred)		
	Frozen		
	Ambient	72 hours	

## **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
Z1123	Lorazepam (Ativan)	Numeric	ng/mL	In Process

**CPT Code:** 1 × 80154

### Reference Values:

Reference Range: 50.0-240.0 ng/mL

Test Performed By: Medtox Laboratories, Inc. 402 W. County Road D St. Paul, MN 55112