

## NEW TEST ANNOUNCEMENT REFERRAL

**NOTIFICATION DATE:** November 27, 2012 **EFFECTIVE DATE:** January 4, 2013

# ANTIDEPRESSANT DRUG SCREEN, UR, QUANTITATIVE Test ID: FASUQ

**EXPLANATION:** The following new orderable, referred to Medtox Laboratories, will be available January 4, 2013.

REFERRAL LAB CODE: 885300

METHODOLOGY: Immunoassay (IA); Liquid Chromatography/Tandem Mass Spectrometry

(LC/MS/MS)

#### **REFERENCE VALUES:**

Antidepressant screen includes the analysis for:

amitriptyline, clomipramine and desmethylclomipramine, cyclobenzaprine, desipramine, doxepin and desmethyldoxepin, fluoxetine and norfluoxetine, imipramine, maprotiline, nortriptyline, paroxetine, protriptyline, sertraline and desmethylsertraline, trimipramine.

### **SPECIMEN REQUIREMENTS:**

Collect 3 mL random urine without preservatives. Ship refrigerated in a plastic container.

### **SPECIMEN STABILITY INFORMATION:**

Specimen Type	Temperature	Time
Urine	Refrigerated (preferred)	14 days
	Ambient	72 hours
	Frozen	180 days

**FEE**: \$132.80

**CPT Code**: 80101; 82542

**DAY(S) SET UP:** Monday through Sunday **ANALYTIC TIME:** 3 days

QUESTIONS: Contact Mary Erath, MML Laboratory Technologist Resource Coordinator Telephone: 800-533-1710