

**TEST UPDATE**

**REFERRAL**

**NOTIFICATION DATE:** March 27, 2013

**EFFECTIVE DATE:** April 1, 2013

**AMYLASE ISOENZYMES**

Test ID: FAMYS

Secondary ID: 57288

**EXPLANATION:** Effective April 1, 2013, Test ID FAMYS, referred to ARUP Laboratories, will reflect the following change.

**CURRENT SPECIMEN REQUIRED:**

**Submit only 1 of the following specimens:**

**Serum**

Draw blood in a plain, red-top tube(s). (Serum gel tube is acceptable.) Spin down and send 1 mL of serum refrigerated in a plastic vial.

**Plasma**

Draw blood in a green-top (heparin) tube(s) or a lavender-top (EDTA) tube(s). Spin down and send 1 mL plasma refrigerated in a plastic vial.

**NEW SPECIMEN REQUIRED:**

**Submit only 1 of the following specimens:**

**Serum**

Draw blood in a plain, red-top tube(s). (Serum gel tube is acceptable.) Spin down and send 1 mL of serum refrigerated in a plastic vial.

**Plasma**

Draw blood in a green-top (sodium or lithium heparin) tube(s). Spin down and send 1 mL plasma refrigerated in a plastic vial.

QUESTIONS: Contact Mary Erath, MML Laboratory Technologist Resource Coordinator  
Telephone: 800-533-1710