



Test Definition: SIIRO

Sirolimus, Blood

Reporting Title: Sirolimus, B **Performing Location:** Rochester

Specimen Requirements:

Container/Tube: Lavender top (EDTA)

Specimen Volume: 3 mL Collection Instructions:

- 1. Draw blood immediately before a scheduled dose.
- 2. Do not centrifuge.
- 3. Send specimen in original tube.

Additional Information:

- 1. Therapeutic range applies to trough specimen drawn immediately prior to a.m. dose.
- 2. For specimens sent ambient, sirolimus analysis cannot be performed; specimen must be sent refrigerated or frozen.

Specimen Type	Temperature	Time
Whole Blood EDTA	Refrigerated (preferred)	7 days
	Frozen	14 days
	Ambient	48 hours

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
35144	Sirolimus, B	Numeric	ng/mL	In Process

CPT Code: 1 × 80195

Reference Values:

4-20 ng/mL