

### **Test Definition: GHIV**

HIV-1 Genotypic Drug Resistance Mutation Analysis, Plasma

Reporting Title: HIV-1 Genotypic Drug Resistance, P

Performing Location: Rochester

### **Specimen Requirements:**

Collection Container/Tube: Lavender top (EDTA)

Submission Container/Tube: Plastic vial

Specimen Volume: 2 mL Collection Instructions:

- 1. Spin down and separate plasma from cells within 6 hours of draw.
- 2. Freeze specimen immediately, and ship frozen on dry ice.
- 3. If shipment will be delayed for >24 hours, freeze specimen at -70 degrees C (up to 35 days) until shipment on dry ice. Additional Information:
- 1. This test is intended to be used to monitor known HIV-positive infections. It is not intended for primary detection of HIV infections.
- 2. Specimens submitted for HIV-1 genotyping should contain > or =1,000 copies/mL of HIV-1 RNA.

Specimen Type	Temperature	Time
Plasma EDTA	Frozen	35 days

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
82340	HIV-1 Genotypic Drug Resistance, P	Alphanumeric		N/A
	Also used by tests: GHIVR			
21395	Reverse Transcriptase Mutations	Alphanumeric		30554-0
	Also used by tests: GHIVR			
21411	Abacavir	Alphanumeric		30287-7
	Also used by tests: GHIVR			
21406	Didanosine	Alphanumeric		30284-4
	Also used by tests: GHIVR			
21412	Lamivudine/Emtricitabine	Alphanumeric		30283-6
	Also used by tests: GHIVR			
21408	Stavudine	Alphanumeric		30286-9
	Also used by tests: GHIVR			



# **Test Definition: GHIV**

HIV-1 Genotypic Drug Resistance Mutation Analysis, Plasma

Result ID	Reporting Name	Туре	Unit	LOINC®
21530	Tenofovir	Alphanumeric		41396-3
	Also used by tests: GHIVR			
21405	Zidovudine	Alphanumeric		30282-8
	Also used by tests: GHIVR			
21414	Efavirenz	Alphanumeric		30291-9
	Also used by tests: GHIVR			
31267	Etravirine	Alphanumeric		52749-9
	Also used by tests: GHIVR			
21410	Nevirapine	Alphanumeric		30289-3
	Also used by tests: GHIVR			
34917	Rilpivirine	Alphanumeric		In Process
	Also used by tests: GHIVR			
21400	Protease Mutations	Alphanumeric		33630-5
	Also used by tests: GHIVR			
23624	Atazanavir	Alphanumeric		41403-7
	Also used by tests: GHIVR			
28076	Atazanavir with Ritonavir	Alphanumeric		49618-2
	Also used by tests: GHIVR			
26784	Darunavir with Ritonavir	Alphanumeric		49630-7
	Also used by tests: GHIVR			
26732	Fosamprenavir	Alphanumeric		43736-8
	Also used by tests: GHIVR			
26733	Fosamprenavir with Ritonavir	Alphanumeric		51409-1
	Also used by tests: GHIVR			
21415	Indinavir	Alphanumeric		30292-7
	Also used by tests: GHIVR			
26734	Indinavir with Ritonavir	Alphanumeric		49619-0
	Also used by tests: GHIVR			
21532	Lopinavir with Ritonavir	Alphanumeric		42000-0
	Also used by tests: GHIVR			



# **Test Definition: GHIV**

HIV-1 Genotypic Drug Resistance Mutation Analysis, Plasma

Result ID	Reporting Name	Туре	Unit	LOINC®
21416	Nelfinavir	Alphanumeric		30294-3
	Also used by tests: GHIVR			
26735	Saquinavir with Ritonavir	Alphanumeric		49621-6
	Also used by tests: GHIVR			
28201	Tipranavir with Ritonavir	Alphanumeric		49622-4
	Also used by tests: GHIVR			

**CPT Code:** 1 × 87901

**Reference Values:** 

Not applicable