



MAYO
Mayo Medical Laboratories
1-800-533-1710

NEW TEST ANNOUNCEMENT

NOTIFICATION DATE: August 26, 2013

EFFECTIVE DATE: August 27, 2013

ANTI-DNASE B TITER, SERUM Test ID: ADNAS

NOTE: The Anti-DNase Titer, serum is currently orderable only as part of the SABP-Streptococcal Antibodies Panel. Effective August 27th, ADNAS will be available as an individually orderable test.

USEFUL FOR: Demonstration of acute or recent streptococcal infection

METHODOLOGY: Nephelometry

REFERENCE VALUES:

Age	Value
< 5 years	< or =250 U/mL
5-17 years	< or = 375 U/mL
> or =18 years	< or = 300 U/mL

SPECIMEN REQUIREMENTS: Container/Tube:

Preferred: Red top

Acceptable: Serum gel

Specimen Volume: 1 mL

Minimum Volume: 0.5mL

NOTE: Fasting preferred but not required

SPECIMEN STABILITY INFORMATION:

Specimen Type	Temperature	Time
Serum	Refrigerated (preferred)	14 days
	Ambient	7 days
	Frozen	14 days

CAUTIONS: False-high titers may be obtained with sera that are contaminated by certain bacterial organisms during shipment or storage and in patients with liver disease where the presence of high lipoprotein concentrations in the serum may mimic antibody activity.

CPT CODE: 86215

DAY(S) SET UP: Monday through Saturday; **ANALYTIC TIME:** Same day/1 day
Continuously until 3pm

QUESTIONS: Contact your Mayo Medical Laboratories' Regional Manager or
Dawn Keller, MML Laboratory Technologist Resource Coordinator
Telephone: 800-533-1710