

**Reporting Title:** Drug of Abuse, Clin, DAU9, U  
**Performing Location:** New England

**Specimen Requirements:**

Container/Tube: Plastic, 60-mL urine container (Supply T313)

Specimen Volume: 30 mL

Collection Instructions:

1. Collect a random urine specimen.
2. No preservative.

Specimen Type	Temperature	Time
Urine	Refrigerated (preferred)	14 days
	Ambient	14 days
	Frozen	14 days

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
11164	Amphetamines <b>Also used by tests: DAU, DAU7, DAAMP, DAU8</b>	Numeric	ng/mL	8150-5
11301	Barbiturates <b>Also used by tests: DABAR, DAU, DAU7, DAU8</b>	Numeric	ng/mL	9426-8
11572	Benzodiazepines <b>Also used by tests: DAU7, DABZO, DAU, DAU8</b>	Numeric	ng/mL	9428-4
11161	Cocaine Metabolite <b>Also used by tests: DAU8, DAU, DAU7, DAUCO</b>	Numeric	ng/mL	3394-4
11165	Methadone <b>Also used by tests: DMETH, DAU</b>	Numeric	ng/mL	3774-7
11570	Opiates <b>Also used by tests: DAU8, DAU7, DAU</b>	Numeric	ng/mL	8220-6
11573	Phencyclidine <b>Also used by tests: DAU7, DAU8, DPCP, DAU</b>	Numeric	ng/mL	3937-0

Result ID	Reporting Name	Type	Unit	LOINC®
11162	Tetrahydrocannabinols <b>Also used by tests: DAU7, DAU8, DTHC, DAU</b>	Numeric	ng/mL	3530-3
200209	Comment <b>Also used by tests: DAU7, DAU8, DAU</b>	Alphanumeric		48767-8

**CPT Code:** 1 x 80104

**Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
MSOP	Drug of Abuse, Opiate Conf, U	1	83925	No	No
DAUAM	Drug of Abuse, Amphetamine Conf, U	2	82145	No	Yes
DAUBZ	Drug of Abuse, Benzo Conf, U	1	80154	No	Yes
MSBR	Drug of Abuse, Barbiturate Conf, U	1	82205	No	Yes
MSCO	Drug of Abuse, Cocaine Conf, U	1	82520	No	Yes
MSMT	Drug of Abuse, Methadone Conf, U	1	83840	No	Yes
MSPCP	Drug of Abuse, Phencyclidine Conf U	1	83992	No	Yes
MSTHC	Drug of Abuse, THC Conf, U	1	80299	No	Yes

**Result Codes for Reflex Tests:**

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
MSOP	11141	Confirmation - Opiates	Alphanumeric		17384-9
MSOP	200210	Codeine	Alphanumeric	ng/mL	16250-3
MSOP	200211	Hydrocodone	Alphanumeric	ng/mL	16252-9
MSOP	200212	Hydromorphone	Alphanumeric	ng/mL	16998-7
MSOP	200214	Morphine	Alphanumeric	ng/mL	16251-1
MSOP	200213	Oxycodone	Alphanumeric	ng/mL	16249-5
MSOP	200289	Oxymorphone	Alphanumeric	ng/mL	17395-5
DAUAM	501207	GC/MS Confirmation - Amphetamine	Alphanumeric		16234-7



Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
DAUAM	501208	Amphetamine	Alphanumeric	ng/mL	16234-7
DAUAM	501209	Methamphetamine	Alphanumeric	ng/mL	16235-4
DAUAM	500197	MDMA (Ecstasy)	Alphanumeric	ng/mL	18358-2
DAUAM	501210	MDA (Ecstasy Metabolite)	Alphanumeric	ng/mL	27409-2
DAUAM	501211	Phentermine	Alphanumeric	ng/mL	20557-5
DAUAM	501212	Pseudoephedrine/Ephedrine	Alphanumeric	ng/mL	26785-6
DAUAM	501213	Phenylpropanolamine	Alphanumeric	ng/mL	17443-3
DAUAM	501214	Comment	Alphanumeric		48767-8
DAUBZ	501215	GC/MS Confirmation - Benzodiazepine	Alphanumeric		20412-3
DAUBZ	501216	Lorazepam	Alphanumeric	ng/mL	17088-6
DAUBZ	501217	Nordiazepam	Alphanumeric	ng/mL	16228-9
DAUBZ	501218	Oxazepam	Alphanumeric	ng/mL	16201-6
DAUBZ	501219	Temazepam	Alphanumeric	ng/mL	20559-1
DAUBZ	501220	OH-Ethyl-Flurazepam	Alphanumeric	ng/mL	20532-8
DAUBZ	501221	7-NH-Clonazepam	Alphanumeric	ng/mL	51776-3
DAUBZ	501222	7-NH-Flunitrazepam	Alphanumeric	ng/mL	51777-1
DAUBZ	501223	Alpha OH-Alprazolam	Alphanumeric	ng/mL	16348-5
DAUBZ	501224	Alpha-OH-Triazolam	Alphanumeric	ng/mL	49876-6
MSBR	501225	Comment	Alphanumeric		48767-8
MSBR	2837	GC/MS Confirmation - Barbiturate	Alphanumeric		20411-5
MSBR	501226	Amobarbital	Alphanumeric	ng/mL	16239-6
MSBR	501227	Butalbital	Alphanumeric	ng/mL	16237-0
MSBR	501228	Pentobarbital	Alphanumeric	ng/mL	16240-4
MSBR	501229	Phenobarbital	Alphanumeric	ng/mL	16241-2
MSBR	501230	Secobarbital	Alphanumeric	ng/mL	16238-8
MSCO	COCAC	GC/MS Confirmation - Cocaine	Alphanumeric		20519-5
MSCO	COCUC	Cocaine	Alphanumeric	ng/mL	20519-5
MSCO	BEUC	Benzoyllecgonine	Alphanumeric	ng/mL	16226-3
MSCO	CMT94	Comment	Alphanumeric		N/A
MSMT	11147	GC/MS Confirmation - Methadone	Alphanumeric		16246-1
MSMT	501240	Methadone	Alphanumeric	ng/mL	16246-1

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
MSMT	501241	Comment	Alphanumeric		48767-8
MSPCP	11146	GC/MS Confirmation - Phencyclidine	Alphanumeric		16254-5
MSPCP	501238	Phencyclidine	Alphanumeric	ng/mL	16254-5
MSPCP	501239	Comment	Alphanumeric		48767-8
MSTHC	11145	GC/MS Confirmation - THC	Alphanumeric		20413-1
MSTHC	501233	THC Carboxylic Acid	Alphanumeric	ng/mL	3530-3
MSTHC	501234	Comment	Alphanumeric		48767-8

### Reference Values:

#### Negative

The specific drug identified will be reported.

EMIT cutoff concentrations

Amphetamines: 500 ng/mL

Barbiturates: 200 ng/mL

Benzodiazepines: 200 ng/mL

Cocaine (benzoylecgonine-cocaine metabolite): 150 ng/mL

Methadone: 300 ng/mL

Opiates: 300 ng/mL

Phencyclidine: 25 ng/mL

THC: 50 ng/mL

Results of this test are confirmed and should be considered definitive. Specimens are normally kept for 2 weeks after Mayo Medical Laboratories New England receipt date.