



**MAYO**  
**Mayo Medical Laboratories**  
**1-800-533-1710**

**NEW TEST ANNOUNCEMENT**  
**REFERRAL**

**NOTIFICATION DATE:** October 17, 2013

**EFFECTIVE DATE:** November 25, 2013

**COCAINE ANALYSIS-WHOLE BLOOD**  
**Test ID: FCABL**

**EXPLANATION:** The following new orderable, referred to Medtox Laboratories, Inc., will be available November 25, 2013. This test will replace test ID – FCOCB.

Note: This change is due to reporting field changes in test build requiring a new test ID.

**REFERRAL LAB CODE:** 9313

**REFLEX TESTS**

Test ID	Reporting Name	Available Separately	Always Performed
FCOCO	Cocaine, Confirmation	No	No

**TESTING ALGORITHM**

If the Cocaine Analysis, whole blood is positive, then the Cocaine, Confirmation (FCOCO) will be performed at no additional charge.

**METHODOLOGY:** Immunoassay (IA)

Gas Chromatography with mass spectrometry (GC/MS); confirmation

**REFERENCE VALUES:**

This specimen was screened by Immunoassay. Any positive result is confirmed by gas chromatography with mass spectrometry (GC/MS). The following threshold concentrations are used for this analysis:

<u>Drug</u>	<u>Screening Threshold</u>	<u>Confirmation Threshold</u>
Cocaine metabolites	25 ng/mL	
Cocaine		30 ng/mL
Benzoyllecgonine		30 ng/mL

**SPECIMEN REQUIREMENTS:**

Draw blood in a purple-top (EDTA), green-top (sodium-heparin) or grey-top (naF/oxalate) tube(s). Send 7 mL of whole blood refrigerated in a plastic vial.

**SPECIMEN STABILITY INFORMATION:**

Specimen Type	Temperature	Time
Whole Blood	Refrigerated (preferred)	14 days
	Frozen	365 days

**FEE:** \$81.60

**CPT CODE:**

80101 - Single drug class method

82542 - Quantitation (if appropriate)

**DAY(S) SET UP:** Monday - Sunday

**ANALYTIC TIME:** 5 days

QUESTIONS: Contact Mary Erath, MML Referrals Supervisor  
Telephone: 800-533-1710