

**SPECIMEN REQUIREMENT
CHANGE****NOTIFICATION DATE:** February 11, 2014**EFFECTIVE DATE:** February 11, 2014**CHLORDIAZEPOXIDE, SERUM**
Test ID: CDP

EXPLANATION: Due to recent specimen stability studies, specimens submitted for Chlordiazepoxide, Serum testing will now require protection from light. Preferred temperature will remain as refrigerated.

CURRENT SPECIMEN REQUIRED	NEW SPECIMEN REQUIRED
Container/Tube: Red top Specimen Volume: 3 mL Collection Instructions: Spin down within 2 hours of draw.	Container/Tube: Red top Submission Container/Tube: Amber vial (Supply T192) Specimen Volume: 3 mL Collection Instructions: Spin down within 2 hours of draw. Send specimen in amber vial to protect from light.

QUESTIONS: Contact your Mayo Medical Laboratories' Regional Manager or
Jim Nielsen, MML Laboratory Technologist Resource Coordinator
Telephone: 800-533-1710